

Ino



Care reclining armchair \

This comfortable reclining armchair was developed specifically as a universal design recliner that evolves with the needs and desires of its user. Because of its modular structure, the reclining armchair can even be converted from a 'design version' to a full 'care version'.

The Ino does not only have a light modern look, it actually is a very light chair. The chair's sleek design gives it a sense of sophistication while also ensuring a high level of stability. The Ino is very manoeuvrable and very maintenance-friendly too.

Our collection of contemporary fabrics adds a personal touch to the chair and ensures that it fits perfectly in your interior. The numerous available options and accessories include a dining tray attachment and coffee table. All in all, it is a unique product! Experience it for yourself!

What makes the Ino Care Relax so unique? \

- \\ Very comfortable
- \\ Ergonomic operation
- \\ Large 'care' handles
- \\ User-friendly
- \\ Maintenance-friendly
- \\ Modular design can be adjusted at any time
- \\ Manoeuvrability and mobility intended to move
- \\ Different seating heights
- \\ Manual head support adjustment integrated as standard
- \\ Extreme stability and strength
- \\ Hoist accessible (active and passive hoist)
- \\ Optionally available as sleeping armchair



OVERVIEW OF INO CARE RELAX



Armrests

Available with 10 different armrests, both open and closed and in different types of materials.

More information on page 5

Adjustment mechanism

4 different adjustment mechanisms are possible, both manual and electric. The Zero Gravity version is available with a relaxing position and also a sleeping position.

More information on pages 3 and 4

Frame & castors

Available with 5 different types of base, depending on the frame, castor size and braking method.


More information on page 7

Options & accessories


The Ino Care Relax has a whole list of options and accessories to adapt the Care Relax to the user's needs.

More information on page 9

RECLINING MECHANISMS

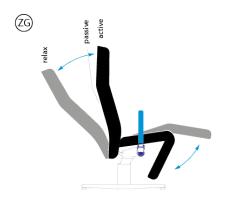


ZERO GRAVITY 125°

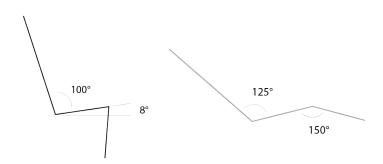


Pure simplicity. The back and leg rest are synchronously steplessly adjustable. From an active or passive sitting posture to an excellent relaxing position.

The maximum angle between the back and the seat is 125° in the relaxing position.

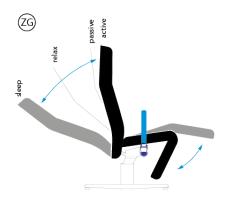


ZERO GRAVITY 140°

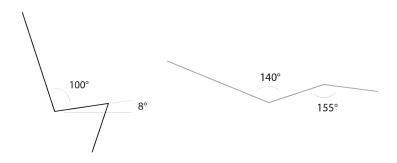

Pure simplicity. The back and leg rest are synchronously steplessly adjustable. From an active or passive sitting posture to an excellent relaxing or sleeping position.


The maximum angle between the back and the seat is 125° in the relaxing position and 140° in the sleeping position.

RECLINING MECHANISMS

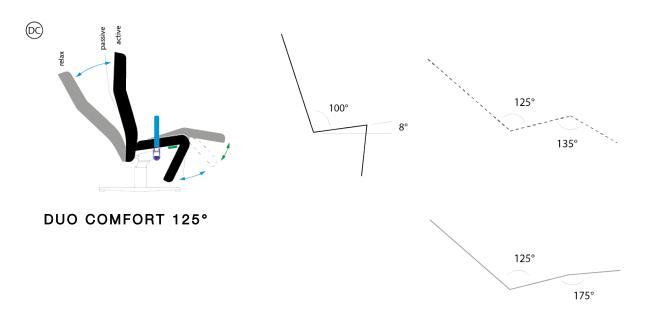

Ergonomics. Duo Comfort offers numerous sitting and lying positions. The back and leg rest are individually steplessly adjustable. From an active and passive sitting posture to an excellent relaxing position.

The maximum angle between the back and the seat is 125° in the relaxing position.

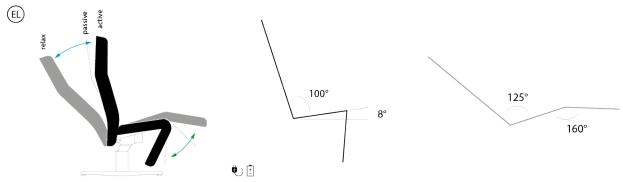

ELECTRICAL 125°


Easy to use. Seat, back and leg rest can be adjusted electrically. 2 motors: 1 for the operation of the back, 1 for the operation of the leg rest. Back and leg rest can be operated together or separately.

The maximum angle between the back and the seat is 125° in the relaxing position.

Possibility of removable and rechargeable battery in docking station.


ARMRESTS

The Ino Care Relax can be fitted with various armrests, ranging from comfortable fully upholstered armrests to slide away armrests. Depending on the type, it may also be possible to attach a tray to the armrest.

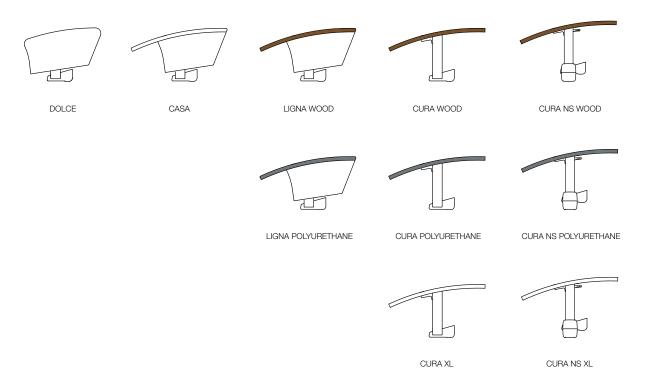


FRAME & CASTORS

Aluminium star base with aluminium column, epoxy-coated. All versions with \emptyset 125 mm castors are provided with an ergonomically-shaped plastic push bar and a foot support. The following versions are possible:

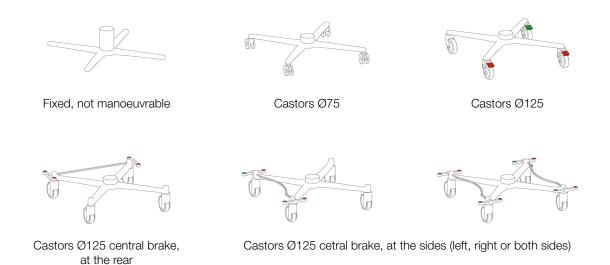

TECHNICAL FEATURES

DIMENSIONS

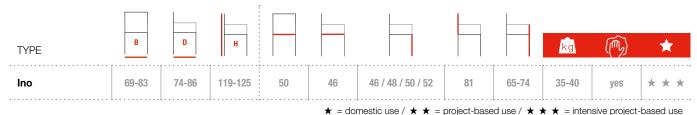

(*) Seat height

Each Ino care reclining armchair is available in 4 seating heights: 46-48-50 and 52 cm.

The seating heights can be adjusted easily with the modular design of the Ino.

(**) Seat width

Measured between the armrests and the seat: front 50 cm - rear 48 cm

Armrest height

21 cm - measured from the seat

SEAT, BACK AND LEG REST

Upholstered. Ergonomically shaped.

Particularly comfortable chair thanks to combination of elastic webbing over a steel tube frame and complete foam padding. The high-density foam which is injected is fire retardant.

The underside of the seat is finished with an elastic, moisture-repellent and washable material as standard. Every chair has a manually adjustable head support as standard.

With the integrated leg rest there are no annoying edges and openings between the seat and leg rest. Functionally designed minimum opening between the seat and back for easy cleaning.


Version with Ø 125 mm castors is provided with an ergonomically-shaped plastic push bar as standard.

OPTIONS AND ACCESSOIRES

There are a number of options and accessories available for the Ino, including a dining tray, serum holder and coffee table,... Take a look at our technical data sheets and brochure to see all possibilities.

USE

Ino Zero Gravity and Ino Duo Comfort each have 2 ergonomically formed plastic control handles. The recliner is simply operated by moving these handles forwards (push) or backwards (pull), this will release the gas spring and enables you to smoothly switch to the desired sitting or lying position. It is possible to lock the reclining armchair in any sitting or lying position of your choice, or to have the reclining armchair move freely.

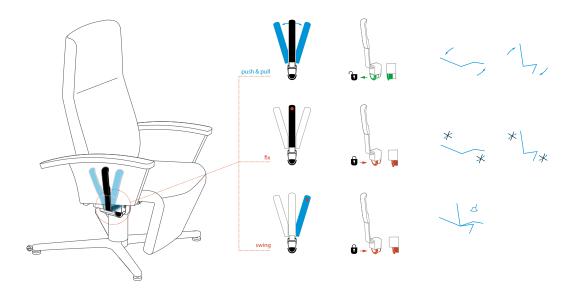

The Ino Electric has a permanent battery as standard (via cable feed). The Ino can be easily placed in the desired sitting or sleeping positions via the simple manual control. There is also the option of working with a removable and rechargeable battery to gain greater freedom of movement within the room.

OVERVIEW OF MODELS



INO DOLCE FIXED



INO DOLCE CASTORS Ø 75



INO DOLCE CASTORS Ø 125



INO DOLCE CASTORS Ø 125 CENTRAL BRAKE



INO CASA FIXED



INO CASA CASTORS Ø 75



INO CASA CASTORS Ø 125



INO CASA CASTORS Ø 125 CENTRAL BRAKE



INO LIGNA FIXED



INO LIGNA CASTORS Ø 75



INO LIGNA CASTORS Ø 125



INO LIGNA CASTORS Ø 125 CENTRAL BRAKE

OVERVIEW OF MODELS



INO CURA FIXED



INO CURA CASTORS Ø 75



INO CURA CASTORS Ø 125



INO CURA CASTORS Ø 125 CENTRAL BRAKE



INO CURA NS FIXED



INO CURA NS CASTORS Ø 75



INO CURA NS CASTORS Ø 125



INO CURA NS CASTORS Ø 125 CENTRAL BRAKE